


CIRCPPLUS™

VEIN PROCEDURE COMPRESSION SYSTEM

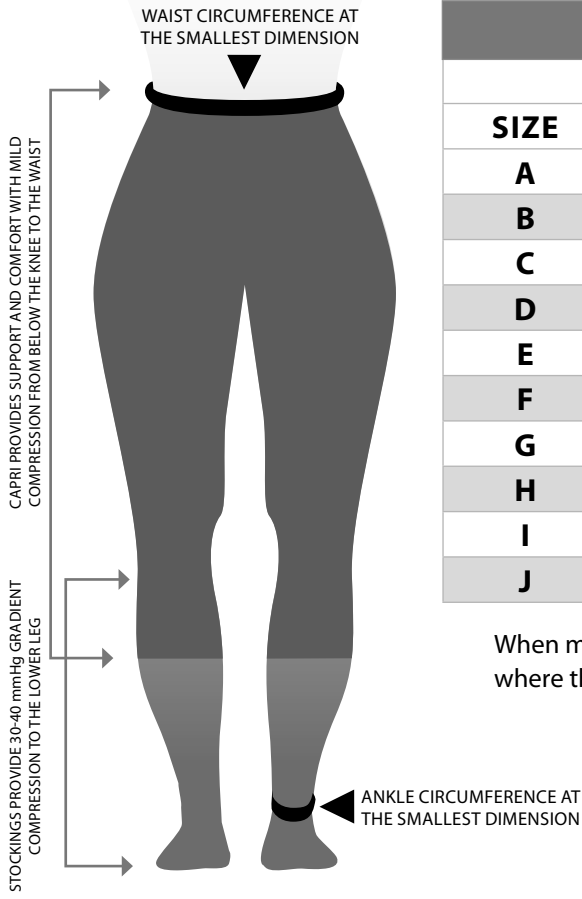

SIZING				
	INCHES		CENTIMETERS	
SIZE	WAIST	ANKLE	WAIST	ANKLE
A	25 - 31	7 - 8	63 - 79	18 - 20
B	26 - 33	7 - 8	67 - 85	18 - 20
C	28 - 36	8 - 9.5	72 - 91	21 - 24
D	30 - 38	8 - 9.5	76 - 96	21 - 24
E	32 - 40	8 - 9.5	81 - 102	21 - 24
F	33 - 43	9.5 - 11	85 - 108	25 - 28
G	35 - 44	9.5 - 11	90 - 113	25 - 28
H	37 - 47	9.5 - 11	94 - 119	25 - 28
I	39 - 49	9.5 - 11	99 - 125	25 - 28
J	41 - 51	9.5 - 11	103 - 130	25 - 28

When measurements fall into multiple sizes, choose the capri size where the waist measurement is in the middle of the range.

PACKAGE CONTENTS:
 One (1) Comfort Capri™
 One (1) Pair of 30-40 mmHg
 CircPlus™ Compression Stockings



EC REP

MDSS
 Schiffgraben 41
 30175 Hannover, Germany



Components of Comfort Capri™: Nylon, Polyester, Lycra®, Cotton. Components of Compression Stocking: Nylon, Spandex. **THESE PRODUCTS DO NOT CONTAIN LATEX.** CircAid Medical Products, Inc. assumes no liability for injury or damages resulting from contraindicated use of its products. CircAid is a registered trademark. Comfort Capri and CircPlus are trademarks of CircAid Medical Products, Inc. Lycra is a registered trademark of INVISTA. ©2012 CircAid Medical Products, Inc. All Rights Reserved.

9323 CHESAPEAKE DRIVE, SUITE B2, SAN DIEGO, CA 92123 | (800) CIRCAID 247-2243 | info@circaid.com | www.circaid.com

CP-IN-112612-01

NOTE: To extend the life of your CircPlus™ System hand wash and drip or flat dry.

- DO NOT IRON
-
- DO NOT DRY CLEAN
-
- TUMBLE DRY LOW HEAT
-
- DO NOT WRING
-
- DO NOT BLEACH
-
- MACHINE WASH COLD
-

CARE INSTRUCTIONS

CIRCPLUS™

VEIN PROCEDURE COMPRESSION SYSTEM

DIRECTIONS FOR USE:

DONNING THE CIRCPLUS™ COMPRESSION SYSTEM:

STEP 1. Roll the top edge of the stocking down until the heel of the stocking is visible. Grip the rolled edge of the stocking and slide the foot into the stocking.

STEP 2. Unroll the rest of the CircPlus™ Compression Stocking onto the leg.

STEP 3. Slide on the Comfort Capri™ and pull it up to the waist. The bottom elastic cuff of the Capri should overlap with the stocking.

REMOVING THE CIRCPLUS™ COMPRESSION SYSTEM:

STEP 1. Slide the Comfort Capri™ off the legs.

STEP 2. Slide the CircPlus™ Compression Stockings off the legs.

Note: *The CircPlus™ Compression System should never hurt. It should feel firm but comfortable. Both garments should be removed at night or while lying down.*

 **If you are experiencing any pain from the garments, immediately remove them and consult your physician.**

Instructional videos are available at: www.circaid.com/circplus

If the compression stockings are hard to don, contact customer service at (800) CIRCAID (247-2243) for more information on alternatives to compression stockings.

