

CIRCPLUS™

VEIN PROCEDURE COMPRESSION SYSTEM

DIRECTIONS FOR USE:

DONNING THE CIRCPLUS™ COMPRESSION SYSTEM:

STEP 1. Roll the top edge of the stocking down until the heel of the stocking is visible. Grip the rolled edge of the stocking and slide the foot into the stocking.

STEP 2. Unroll the rest of the CircPlus™ Compression Stocking onto the leg.

STEP 3. Slide on the Comfort Capri™ and pull it up to the waist. The bottom elastic cuff of the Capri should overlap with the stocking.

REMOVING THE CIRCPLUS™ COMPRESSION SYSTEM:

STEP 1. Slide the Comfort Capri™ off the legs.

STEP 2. Slide the CircPlus™ Compression Stockings off the legs.



Note: The CircPlus™ Compression System should never hurt. It should feel firm but comfortable. Both garments should be removed at night or while lying down.

 **If you are experiencing any pain from the garments, immediately remove them and consult your physician.**

If the compression stockings are hard to don, contact customer service at (800) CIRCAID (247-2243) for more information on alternatives to compression stockings.

Instructional videos are available at: www.circaid.com/circplus



  CircAid Medical Products, Inc.
9323 Chesapeake Drive, Suite B2
San Diego, CA 92123 U.S.A.
01-858-576-3550

CIRCAID
by medi

EC REP

MDS
Schiffgraben 41
30175 Hannover, Germany