

## CARE AND MAINTENANCE



HAND WASH



DRIP OR LINE DRY



DO NOT MACHINE WASH



DO NOT BLEACH



DO NOT IRON



DO NOT DRY CLEAN



DO NOT WRING

The Comfort Non-Slip Liner™ should be hand washed and air dried. Due to the nature of the material, debris will adhere to it; use tweezers to remove debris that does not wash out. Note that debris will decrease the effectiveness of the material over time.



**This product should not be used by children six years of age and younger.**



# COMFORT NON-SLIP LINER™

The Comfort Non-Slip Liner™ is worn directly against the skin, underneath any compression legging. Its tacky surface aids in preventing the garment from sliding.

COMPONENTS OF COMFORT NON-SLIP LINER™:  
Polyester fiber with binders.


**THIS PRODUCT DOES NOT CONTAIN LATEX.**

CircAid Medical Products, Inc. assumes no liability for injury or damages resulting from contraindicated use of its products. CircAid is a registered trademark. Comfort Non-Slip Liner is a trademark of CircAid Medical Products, Inc. ©2012 CircAid Medical Products, Inc. All Rights Reserved.

**CIRCAID**<sup>®</sup>  
by medi



CircAid Medical Products, Inc.  
9323 Chesapeake Drive, Suite B2  
San Diego, CA 92123 U.S.A.  
01-858-576-3550

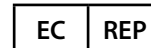


MDSS  
Schiffgraben 41  
30175 Hannover, Germany

# DIRECTIONS FOR USE


## TRIMMING THE COMFORT NON-SLIP LINER™

(For use with your Juxta-Fit™ Legging)


**STEP 1** Unfold the Comfort Non-Slip Liner™ and place it flat on a surface. *Be sure to choose a clean surface as the material attracts debris.* Unroll the tab ends of the legging. Place the legging, or portion of the legging (typically the thigh portion), flat on top of the Comfort Non-Slip Liner™ with the black side of the material facing up.

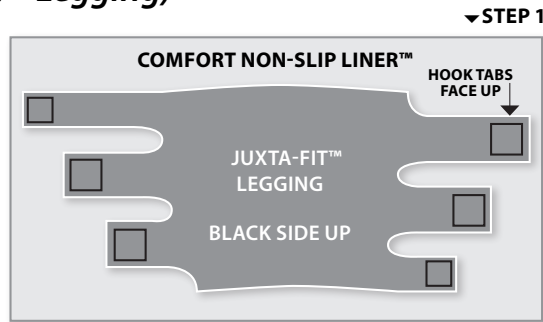

*Note: Do not let the hook tabs come in contact with the Comfort Non-Slip Liner™. The hook tabs may damage the Comfort Non-Slip Liner™ and the fibers of the Comfort Non-Slip Liner™ will decrease the effectiveness of the hook tabs.*

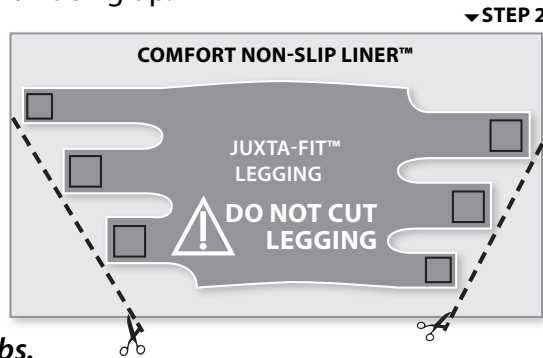


**STEP 2** Using the legging as a template, cut a straight line through the Comfort Non-Slip Liner™. Use the outside edge of the tab ends as a guide.

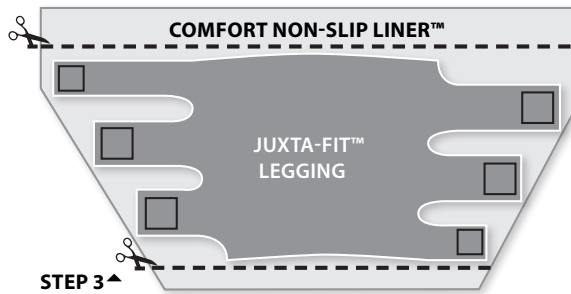

**BE SURE NOT TO CUT THE LEGGING, AS THIS WILL AUTOMATICALLY VOID THE LEGGING'S WARRANTY.**

**STEP 3** If needed, trim the Comfort Non-Slip Liner™ along the top and bottom edge of the legging to minimize excess material.

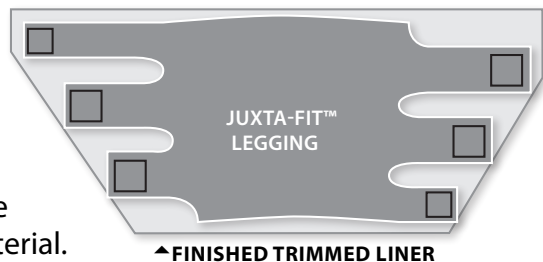

## TRIMMING THE COMFORT NON-SLIP LINER™


(Using Circumference Measurements)

**STEP 1** Measure the height of the desired coverage area on the limb.

*Note: The maximum height of coverage area is 46cm.*

**STEP 2** Measure the largest and bottom circumference of the limb in the coverage area. Add 25cm to each of these measurements to ensure your Comfort Non-Slip Liner™ overlaps.

**STEP 3** Using these three measurements, mark and cut the Comfort Non-Slip Liner™ per the drawing below.

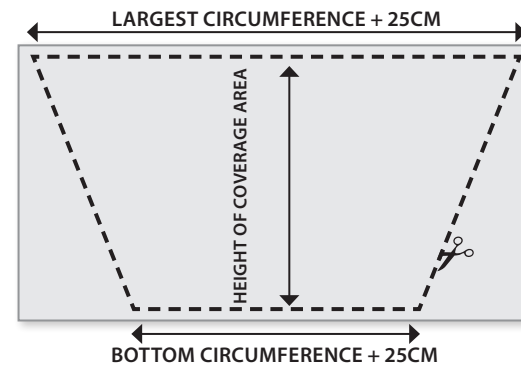

COMFORT NON-SLIP LINER™ WITH JUXTA-FIT™ LEGGING

## DONNING THE COMFORT NON-SLIP LINER™

**STEP 1** Wrap the Comfort Non-Slip Liner™ around the part of the limb where the legging will be donned (e.g. thigh). The Comfort Non-Slip Liner™ should be worn directly against skin; it will not be effective if worn over sock liners.

**STEP 2** Overlap the excess material at the front of the limb to anchor the Comfort Non-Slip Liner™ in place. The Comfort Non-Slip Liner™ will adhere to itself; no tape or other adhesives are necessary.

**STEP 3** Don the legging over the Comfort Non-Slip Liner™. Be sure that none of the hook tabs are adhered to the Comfort Non-Slip Liner™. The hook tabs should be entirely secured to the garment. Getting the hook tabs tangled in the Comfort Non-Slip Liner™ may damage the Comfort Non-Slip Liner™ as well as decrease the effectiveness of the hook tabs.



If further assistance is required, please contact CircAid® Customer Service by phone at (800) CIRCAID (247-2243) or by email at [info@circaid.com](mailto:info@circaid.com).