



medi

PACKAGE INCLUDES:

One (1) Comfort PAC Band™

CARE INSTRUCTIONS:



MACHINE WASH – WARM*



TUMBLE DRY – LOW HEAT*



DO NOT BLEACH



DO NOT IRON



DO NOT DRY CLEAN



DO NOT WRING

*The Comfort PAC Band™ can be machine washed with mild detergent using a gentle cycle and machine dried on permanent press or a low heat setting. It is recommended that it be washed and dried by itself or in a pillowcase to prevent lint from getting caught in the hook tabs. Should lint collect on the tabs, it can be removed with tweezers. To extend the life of the Comfort PAC Band™ hand wash and drip or flat dry.



MDSS
Schiffgraben 41
30175 Hannover, Germany

C-PACB-DFU-111512-01

COMFORT

PAC BAND™

(POWER ADDED COMPRESSION™)

COMPONENTS OF COMFORT PAC BAND™: Nylon, Polyethylene Plastic, Polyurethane, Spandex

LABELS: Polyvinyl Chloride Plastisol

THIS PRODUCT DOES NOT CONTAIN LATEX.

INDICATIONS FOR USE: Lymphedema, Other Forms of Edema: Venous, Post Traumatic, Post Surgical, "Lipedema", Varicose veins (all types), Chronic Venous Insufficiency ("CVI"), Venous Stasis Disease, Venous Valvular Insufficiency ("VVI"), Venous Insufficiency, Post Thrombotic Syndrome, Venous Ulcer (Stasis Ulcer), Angio dysplasia, Venous Eczema, Post Sclerotherapy, Thrombosis Prevention, Lipodermatosclerosis.

CAUTIONS: Decreased or absent sensation in the leg, Sensitivity to product components.

CONTRAINDICATIONS: Severe Peripheral Arterial Disease, Decompensated Congestive Heart Failure, Phlegmasia Cerula Dolens, Septic Phlebitis, Decreased or absent sensation in the leg, Allergy to compression material, Moderate peripheral arterial disease, Infection in the leg.

This garment is not designed to be cut or modified, except as outlined in the directions for use; doing other than instructed will void the 90 day limited warranty.

CircAid Medical Products, Inc. assumes no liability for injury or damages resulting from contraindicated use of its products. CircAid is a registered trademark. Comfort PAC Band, Power Added Compression, and Comfort Compression Anklets are trademarks of CircAid Medical Products, Inc. ©2012 CircAid Medical Products, Inc. All Rights Reserved.



CIRCAID[®]
by medi



CircAid Medical Products, Inc.
9323 Chesapeake Drive, Suite B2
San Diego, CA 92123 U.S.A.
01-858-576-3550

DIRECTIONS FOR USE:

The Comfort PAC Band™ (Power Added Compression™) is designed to provide additional compression to the foot, targeting the arch and ankle area regions. For complete compression of the foot, it is recommended to apply the Comfort PAC Band™ over the Comfort Compression Ankle™.



STEP 1: Lay the Comfort PAC Band™ out flat. If using on right foot, the right foot emblem should be visible. If using on the left foot, the left foot emblem should be visible.




STEP 2: Center arch of foot on footprint and wrap arch strap with firm and comfortable compression. Secure band with the large hook tab.

STEP 3: Pull remaining band around the back of foot with firm and comfortable compression. Secure band with the small hook tab.



STEP 4: If excess material or overlap is a concern, either the arch or ankle band may be trimmed as needed.

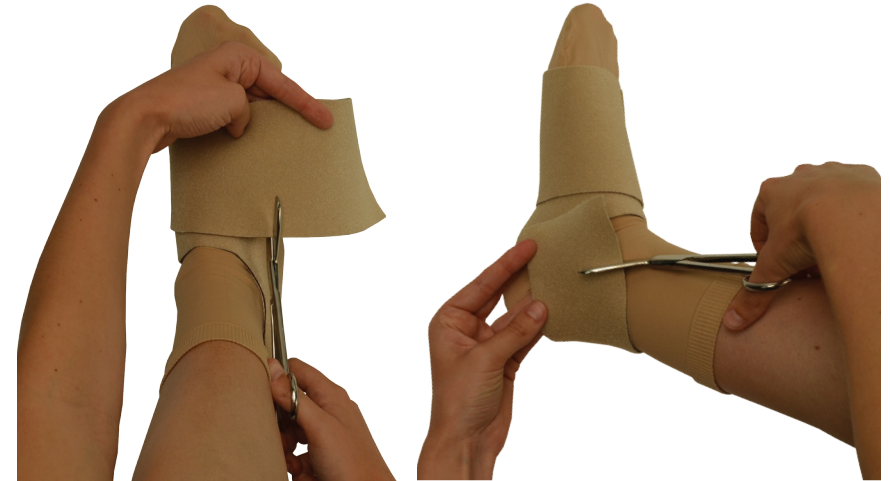

REMOVAL:

Disengage the hook tabs and fold each band back onto the body of the garment. Do not tightly roll the band back onto itself as this will cause the hook material to bend, reducing its adherence strength. When complete, the garment should be ready for easy reapplication.



REMEMBER: The Comfort PAC Band™ should never hurt. It should feel firm but comfortable. If pressure increases or decreases noticeably during wear, readjust the Comfort PAC Band™ accordingly.

IF EXPERIENCING ANY PAIN, REMOVE THE COMFORT PAC BAND™ IMMEDIATELY AND CONSULT YOUR PHYSICIAN.

CIRCAID
by medi

If further assistance is required, contact CircAid® Customer Service by phone at (800) CIRCAID (247-2243) or email at info@circaid.com