



medi

## THIS PACKAGE INCLUDES:

- (1) Graduate™ Custom Boot
- (1) Pair of Comfort Knee-High Socks™

## THIS PACKAGE MAY ALSO INCLUDE:

- (1) Graduate™ Non-Skid Foot Guard
- (1) Graduate™ Rubber Sole (Tread)

If any of these items are missing, or replacement Graduate™ Non-Skid Foot Guards and/or Rubber Soles are required, please contact Customer Service at (800)-CIRCAID (247-2243) or by email at [info@circaid.com](mailto:info@circaid.com). Instructional videos are available online at [www.circaid.com](http://www.circaid.com).

## CARE AND MAINTENANCE



HAND WASH



DRIP DRY OR FLAT DRY



DO NOT BLEACH



DO NOT IRON



DO NOT DRY CLEAN



DO NOT WRING

G-B-DFU-112612-01

# GRADUATE™ CUSTOM BOOT

**COMPONENTS OF GRADUATE™:** Nylon, Polyethylene Plastic, Cotton, Lycra®, Polyester, Neoprene, Polyurethane Foam, Steel (D-Rings)

**COMPONENTS OF COMFORT KNEE-HIGH SOCKS™:** Cotton, Lycra®

**THIS PRODUCT DOES NOT CONTAIN LATEX.**

**INDICATIONS FOR USE:** Lymphedema, Venous Insufficiency, Venous Stasis Ulcers, Post Thrombotic Syndrome, Dependent Edema.

**CAUTIONS:** Decreased or absent sensation in the leg, Sensitivity to product components.

**CONTRAINDICATIONS:** Severe Peripheral Arterial Disease, Septic Phlebitis, Decompensated Congestive Heart Failure, Phlegmasia Cerula Dolens, Allergy to compression materials, Moderate peripheral arterial disease, Infection in the ankle and/or foot.

Custom garments require a prescription. This garment is not designed to be cut or modified in anyway. Doing so will void the one year limited warranty.

CircAid Medical Products, Inc. assumes no liability for injury or damages resulting from contraindicated use of its products. CircAid is a registered trademark. Graduate and Comfort Knee-High Socks are trademarks of CircAid Medical Products, Inc. Lycra is a registered trademark of INVISTA.


©2012 CircAid Medical Products, Inc. All Rights Reserved.



CIRCAID®  
by medi



CircAid Medical Products, Inc.  
9323 Chesapeake Drive, Suite B2  
San Diego, CA 92123 U.S.A.  
01-858-576-3550

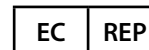

MDSS  
Schiffgraben 41  
30175 Hannover, Germany

# DIRECTIONS FOR USE

## DONNING THE GRADUATE™ CUSTOM BOOT

**STEP 1:** Don enclosed sock liner.

**STEP 2:** Secure the foot straps at their loosest position. Slide the foot into the top opening like a slipper, and firmly press the heel into the heel cup.



**STEP 3:** Ensure that the foot is properly aligned within the garment; the front edge of the Graduate™ Boot should align with the base of the toes. Fasten the hook tab on the foam padding to secure foot placement.



**STEP 4:** Detach the strap nearest to the toes, pull the strap up and across the foot, and fasten to the Graduate™ Boot.



**STEP 5:** Detach the middle strap, pull the strap up and across the foot in the opposite direction as the first strap, and fasten it to the Boot.



**STEP 6:** Lift up the strap closest to the ankle, pull the strap up and across the foot in the same direction as the first strap, and fasten it to the Graduate™ Boot.

**STEP 7:** Readjust the bands as necessary throughout the day to maintain a firm and comfortable compression level.

## REMOVAL

**STEP 1:** Disengage all hook tabs of the Graduate™ Boot and loosen the straps.



**STEP 2:** Refasten the hook tabs at their loosest position.



**STEP 3:** Slide the Graduate™ Boot off the foot.

**STEP 4:** Slide the liner off the leg.



**REMEMBER:** The Graduate™ Custom Boot should never hurt. It should feel firm but comfortable. If pressure increases or decreases noticeably during wear, loosen or tighten any bands accordingly.



**IF EXPERIENCING ANY PAIN,  
IMMEDIATELY REMOVE THE BOOT  
AND CONSULT YOUR PHYSICIAN.**

**CIRCAID**  
by medi

If further assistance is required, contact CircAid® Customer Service by phone at (800) CIRCAID (247-2243) or by email at [info@circaid.com](mailto:info@circaid.com).