medi

DANSK

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ENGLISH

This garment is not designed to be cut or modified in anyway. Doing so will void the six month limited warranty. CircAid Medical Products, Inc. assumes no liability for injury or damages resulting from contraindicated use of its products. CircAid is a registered trademark and Juxta-Fit is a trademark of CircAid Medical Products, Inc. Velcro is a registered trademark of Velcro USA, Inc. ©2012 CircAid Medical Products, Inc. All Rights Reserved.

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CIRCAID®
by medi

CircAid Medical Products, Inc.
9323 Chesapeake Drive, Suite B2
San Diego, CA 92123 U.S.A.
01-858-576-3550

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Juxta-Fit™ Removal Band

DIRECTIONS FOR USE:

The following instructions are an alternative method to removing the Juxta-Fit™ Glove using the enclosed Juxta-Fit™ Removal Band.

STEP 1 Place one of the hook tabs on the top of the fingers so that it is centered over the middle, also insuring good contact so that the hook does not slide off. (It is very important that the hook tabs are attached to the fingers and not on the body of the hand. This method will not work unless done properly)

STEP 2 Do the same thing with the other hook tab, this time firmly securing it to the bottom side of your fingers.

STEP 3 Once secure, gently pull on the band portion of the shelf strap and the glove should easily slip off.

Note: Insure that the wrist band on the glove is detached and rolled back on itself prior to pulling the glove off. Also avoid hook contact with the skin.

CARE INSTRUCTIONS:

-  HAND WASH
-  DRIP DRY OR FLAT DRY
-  DO NOT BLEACH
-  DO NOT IRON
-  DO NOT DRY CLEAN
-  DO NOT WRING

NOTE: To extend the life of your Juxta-Fit™ Removal Band hand wash and drip or flat dry. The Juxta-Fit™ Removal Band can also be machine washed using a gentle cycle and dried on a low heat setting.

COMPONENTS OF JUXTA-FIT™ REMOVAL BAND:
Nylon, Polyethylene Plastic, Polyurethane, Lycra®, Silver

LABELS: Satin Acetate

THIS PRODUCT CONTAINS SILVER.
THIS PRODUCT DOES NOT CONTAIN LATEX.

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