

medi

COMFORT COMPRESSION ANKLET™

STANDARD SIZE
ONE PAIR

ITEM# DT100SA

Fits Arch Circumference Range: 19-36cm

Fits Ankle Circumference Range: 19-27cm

COMPONENTS OF COMFORT COMPRESSION ANKLET™: Nylon and Spandex

THIS PRODUCT DOES NOT CONTAIN LATEX.

INDICATIONS FOR USE: Lymphedema, other forms of Edema: Venous, Post Traumatic, Post Surgical, "Lipedema", Varicose Veins (all types), Chronic Venous Insufficiency ("CVI"), Venous Stasis Disease, Venous Valvular Insufficiency ("VVI"), Venous Insufficiency, Post Thrombotic Syndrome, Venous Ulcer (Stasis Ulcer), Angiodysplasia, Post Sclerotherapy, Thrombosis Prevention, Venous Eczema, Lipodermatosclerosis

CONTRAINDICATIONS: Severe Peripheral Arterial Disease, Decompensated Congestive Heart Failure, Septic Phlebitis, Phlegmasia Cerula Dolens, Decreased or absent sensation in the leg, Allergy to compression material, Moderate peripheral arterial disease, Infection in the leg

CircAid Medical Products, Inc. assumes no liability for injury or damages resulting from contraindicated use of its products. CircAid is a registered trademark and Comfort Compression Anklelet is a trademark of CircAid Medical Products, Inc. ©2012 CircAid Medical Products, Inc. All Rights Reserved.

EC REP

MDSS
Schiffgraben 41
30175 Hannover, Germany



CircAid Medical Products, Inc.
9323 Chesapeake Drive, Suite B2
San Diego, CA 92123 U.S.A.
01-858-576-3550

CIRCAID
by medi

C-CA-S-UK-DFU-112712-01

DIRECTIONS FOR USE:

DONNING THE COMFORT COMPRESSION ANKLET™:

OPTION 1: Slide the Comfort Compression Anklelet™ onto the foot and ensure that it is pulled high enough above the ankle bone so that it will overlap with the CircAid® legging.



OPTION 2: If using a leg liner, apply the liner first. Pull the top part of the Comfort Compression Anklelet™ over the bottom part of the liner. Ensure there are no wrinkles in the liner where it is tucked into the anklelet.



OPTION 3: If irritation to the anklelet's cuffed edge is a concern, pull the top edge of the anklelet over the bottom part of the legging after the legging has been donned.



REMEMBER: The Comfort Compression Anklelet™ should never hurt. It should feel firm but comfortable. The Comfort Compression Anklelet™ may be worn day and night, but removal is recommended if experiencing discomfort when lying down.



IF EXPERIENCING ANY PAIN, IMMEDIATELY REMOVE THE ANKLET AND CONSULT YOUR PHYSICIAN.

REMOVING THE COMFORT COMPRESSION ANKLET™:

Slide the Comfort Compression Anklelet™ off the foot.

CARE INSTRUCTIONS:

- MACHINE WASH WARM
- DO NOT IRON
- TUMBLE DRY LOW
- DO NOT BLEACH

The Comfort Compression Anklelet™ can be machine washed with mild detergent using a gentle cycle and machine dried on permanent press or a low heat setting. To extend the life of the Comfort Compression Anklelet™ hand wash and drip or flat dry.