STØRRELSE / GRÖBE / SIZE / TAILLE / TAGLIA / GROOTTE		
	SMALL (cm)	LARGE (cm)
ARMSLEEVE COMFORT COVERUP™		
A	26cm - 39cm	40cm - 60cm
LOWER LEG COMFORT COVERUP™		
B	34cm - 50cm	51cm - 80cm
WHOLE LEG COMFORT COVERUP™		
C	50cm - 71cm	72cm - 108cm

GB: Take measurements while wearing your compression garment to ensure the best fit.
NL: Neem uw maat op terwijl u het verband draagt om de juiste pasvorm te garanderen.
DA: Tag mål, mens du bærer din kompressionsbeklædning, for at sikre den bedste pasform.
I: Prendere le misurazioni quando indossate l'indumento di compressione per garantire la taglia ottimale.
D: Nehmen Sie die Messungen nur vor, wenn Sie Ihre Kompressionsstrümpfe tragen, um den für Sie komfortabelsten Sitz zu eruieren.
F: Prenez vos mesures alors que vous portez votre orthèse de contention afin de garantir le meilleur ajustement possible.

DANSK

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ENGLISH

Components of Comfort CoverUp™: Nylon, Spandex. THIS PRODUCT DOES NOT CONTAIN LATEX. The Comfort CoverUp™ does not provide therapeutic compression. CircAid Medical Products, Inc. assumes no liability for injury or damages resulting from contraindicated use of its products. CircAid is a registered trademark. Comfort CoverUp is a trademark of CircAid Medical Products, Inc. ©2012 CircAid Medical Products, Inc. All Rights Reserved.

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CIRCAID[®]
by medi

CircAid Medical Products, Inc.
9323 Chesapeake Drive, Suite B2
San Diego, CA 92123 U.S.A.
01-858-576-3550

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Comfort CoverUp™

DIRECTIONS FOR USE:

DONNING Ensure that all tab ends of your compression garment are securely fastened to the garment. No hook tabs should be sticking off the garment. Slide the Comfort CoverUp™ over your compression garment.

REMOVAL To remove the Comfort CoverUp™, slide it off your arm or leg before removing your compression garment.

CARE INSTRUCTIONS

 MACHINE WASH WARM

 TUMBLE DRY LOW

 DO NOT BLEACH

 DO NOT IRON

To extend the life of your Comfort CoverUp™ hand wash and drip or flat dry. The Comfort CoverUp™ can also be machine washed using a gentle cycle and dried on a low heat setting.

It is recommended that the Comfort CoverUp™ be washed inside of a bag or pillowcase to prevent other items from snagging on the Comfort CoverUp™.

The Comfort CoverUp™ can be worn over any compression garment. Its sleek design prevents your compression garments from irritating other parts of your body or snagging on surrounding fabrics. The Comfort CoverUp™ also fits well under clothing. Components of Comfort CoverUp™: Nylon, Spandex. THIS PRODUCT DOES NOT CONTAIN LATEX.

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