

JUXTA-FIT™ DORSUM STRAP

COMPONENTS OF JUXTA-FIT™ DORSUM STRAP:

Nylon, Polyurethane, Spandex, Silver, Polyethylene Plastic

COMPONENTS OF LABEL: Polyvinyl Chloride Plastisol or Satin Acetate

THIS PRODUCT CONTAINS SILVER.

THIS PRODUCT DOES NOT CONTAIN LATEX.

INDICATIONS FOR USE: Lymphedema.

CAUTIONS: Decreased or absent sensation in the hand, Sensitivity to product components.

CONTRAINDICATIONS: Severe arterial insufficiency, acute infection.

CircAid Medical Products, Inc. assumes no liability for injury or damages resulting from contraindicated use of its products. CircAid is a registered trademark and Juxta-Fit is a trademark of CircAid Medical Products, Inc. All Rights Reserved.

©2012 CircAid Medical Products, Inc.



CircAid Medical Products, Inc.
9323 Chesapeake Drive, Suite B2
San Diego, CA 92123 U.S.A.
01-858-576-3550



MDSS
Schiffgraben 41
30175 Hannover, Germany

CIRCAID™
by medi

JF-DS-DFU-112612-01

DIRECTIONS FOR USE:

DONNING THE JUXTA-FIT™ DORSUM STRAP:

STEP 1. Apply Juxta-Fit™ Glove as directed.

STEP 2. Attach the top and bottom hook tabs of the crescent shaped side of the Dorsum Strap above and below the thumb and firmly secure.

STEP 3. Pull the Dorsum Strap across the back of the hand and secure the remaining three hook tabs along the pinky finger side of the glove once firm and comfortable compression is achieved.

NOTE: If desired, the position of the Dorsum Strap can be modified to target specific areas of the hand. To do so, detach the Dorsum Strap hook tabs from the glove, reposition the Dorsum Strap to the preferred location, and then reapply the hook tabs as directed in Steps 2 & 3.

For example, if additional compression is desired along the base of the fingers simply detach the Dorsum Strap hook tabs, slide the strap toward the fingers, and secure the Dorsum Strap hook tabs in the new location, so that added compression is applied.

REMOVING THE JUXTA-FIT™ DORSUM STRAP:

STEP 1. Detach the three Dorsum Strap hook tabs from the pinky side of the hand.

Note: For ease of reapplication it is recommended that the remaining two Dorsum Strap hook tabs secured above and below the thumb remain in place while the glove is removed. Once removed simply reapply the loose hook tabs to the glove to aid in keeping the hook tabs free of debris.

STEP 2. To fully remove the Dorsum Strap remove the remaining two hook tabs from above and below the thumb.

STEP 3. Take the side of the Dorsum Strap with three hook tabs and secure the top and bottom hook tabs on the opposing side, keeping lint from collecting on the hook tabs.

Note: Do not tightly roll the hook tabs. This will cause the tabs to curl and to no longer lay flat on the glove while in use.

CARE INSTRUCTIONS:



HAND WASH - WARM*



DRIP DRY OR FLAT DRY*



DO NOT BLEACH



DO NOT IRON



DO NOT DRY CLEAN



DO NOT WRING

*To extend the life of your Juxta-Fit™ Dorsum Strap hand wash and drip or flat dry. The Juxta-Fit™ Dorsum Strap can also be machine washed using a gentle cycle and dried on a low heat setting.



STEP 1



STEP 2



STEP 3



FINISHED BACK



FINISHED PALM