medi

DANSK

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ENGLISH

This garment is not designed to be cut or modified in anyway. Doing so will void the six month limited warranty. CircAid Medical Products, Inc. assumes no liability for injury or damages resulting from contraindicated use of its products. CircAid is a registered trademark and Juxta-Fit is a trademark of CircAid Medical Products, Inc. Velcro is a registered trademark of Velcro USA, Inc. ©2012 CircAid Medical Products, Inc. All Rights Reserved.

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CIRCAID®
by medi

CircAid Medical Products, Inc.
9323 Chesapeake Drive, Suite B2
San Diego, CA 92123 U.S.A.
01-858-576-3550

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DIRECTIONS FOR USE:

DONNING THE JUXTA-FIT™ GLOVE:

STEP 1 Once your compression garment is in it's proper position and secured, slide the Juxta-Fit™ Glove over the hand.

STEP 2 Ensure the cuff portion is pulled over the wrist section of the compression garment.

STEP 3 Fasten the wrist band on the glove. It should be snugly attached with very light compression.

NOTE: *It is important not to over tighten the glove wrist band so that fluid can freely move up and away from the hand.*

REMOVING THE JUXTA-FIT™ GLOVE:

STEP 1 To remove the glove, first detach the wrist band and fold it back onto its self.

STEP 2 Working with one finger at a time, slowly work them out of the glove by pulling on the edge of each finger. Continue until the glove easily slides off the hand.

Note: *The Juxta-Fit™ Glove should never hurt. It should feel firm but comfortable. If pressure increases or decreases noticeably during wear, loosen or tighten the bands accordingly.*

IF EXPERIENCING ANY PAIN, IMMEDIATELY REMOVE THE JUXTA-FIT™ GLOVE AND CONSULT YOUR PHYSICIAN.

CARE INSTRUCTIONS:

HAND WASH

DRIP DRY OR FLAT DRY

DO NOT BLEACH

DO NOT IRON

DO NOT DRY CLEAN

DO NOT WRING

NOTE: To extend the life of your Juxta-Fit™ Glove hand wash and drip or flat dry. The Juxta-Fit™ Glove can also be machine washed using a gentle cycle and dried on a low heat setting.

COMPONENTS OF JUXTA-FIT™ GLOVE:

Nylon, Polyurethane, Spandex, Silver LABEL: Satin Acetate

THIS PRODUCT CONTAINS SILVER.

THIS PRODUCT DOES NOT CONTAIN LATEX.

INDICATIONS FOR USE: Lymphedema

CONTRAINDICATIONS: Severe arterial insufficiency, Acute infection

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CircAid Medical Products, Inc. påtager sig intet ansvar for uheld eller skader, der opstår fra kontraindiceret brug af vores produkter. CircAid er et registreret varemærke og Juxta-Fit er et varemærke tilhørende CircAid Medical Products, Inc. Velcro er et registreret varemærke tilhørende Velcro USA, Inc. ©2012 CircAid Medical Products, Inc. All Rights Reserved.